

FORUM

The Stiegmeyer-Group's company magazine



The bed extension of the Venta care bed

20. December 2018 // **Did you know?**

Tall residents in nursing homes benefit greatly from a bed extension. In our low-height bed Venta a integrated bed extension of 20 cm is included as standard. Product Manager Michaela Amedick shows you how easy it is to activate this extension without tools.

The bed extension of the Venta care bed

Tall residents in nursing homes benefit greatly from a bed extension. In our low-height bed Venta a integrated bed extension of 20 cm is included as standard. Product Manager Michaela Amedick shows you how easy it is to activate this extension without tools.



Product Manager Michaela Amedick

The bed extension of the low-height bed Venta can be pulled out easily without tools.



Did you know?

First pull down the two knobs underneath the foot section and loosen them with a quarter turn until they click into place.



Did you know?

Now you can grab the footboard and pull out the bed extension. The telescopic side panels are optional accessories.



Did you know?

Turn the two knobs back a quarter turn.



Did you know?

Then slowly push and pull the footboard back and forth until the knobs click back into place. Now the bed extension is locked and can be used.



Did you know?