

FORUM

The Stiegemeyer-Group's company magazine



Adjustment of the foot elevation with the handset at the care bed Regia

29. January 2019 // **Did you know?**

Many users of our care beds very much appreciate adjusting the foot elevation from time to time, bringing their lower legs into a raised, horizontal position. Our product manager Frank Himmelskötter will show you how to adjust this position on the Regia care bed by Burmeier in just a few steps with the handset.

Adjustment of the foot elevation with the handset at the care bed Regia

Many users of our care beds very much appreciate adjusting the foot elevation from time to time, bringing their lower legs into a raised, horizontal position. Our product manager Frank Himmelskötter will show you how to adjust this position on the Regia care bed by Burmeier in just a few steps with the handset.



Did you know ...?




Did you know ...?

1) First lift the thigh rest. To do this, press the left button on the second row on the handset.



Did you know ...?

2) The thigh rest lifts, the lower leg rest remains lowered towards the foot end.

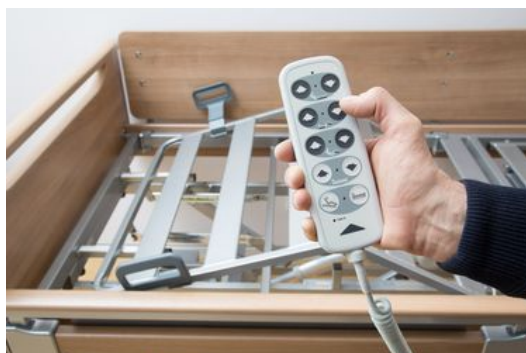

Did you know ...?

3) Now move the thigh rest down again. To do this, press the right button in the 2nd row on the handset. The further you lower the thigh rest, the more level is the activation of the lower leg rest in the next step. But do not bring the mattress base completely into the flat position again.



Did you know ...?

4) Lift the thigh rest again. To do this, press the left button on the second row on the handset.



Did you know ...?

5) Under the lower leg rest, the two ratchets are automatically activated.



Did you know ...?

6) The lower leg rest raises as well and moves to the foot elevation position. If you want to deactivate this position, simply lower the thigh rest until the mattress base is level.
<https://www.stiegmeyer-forum.com/en/did-you-know/adjustment-of-the-foot-elevation-with-the-handset-at-the-care-bed-regia.html>